

Discharge Instructions

Care of your **Mepilex® Ag** Dressing

Your wound has been dressed with a silicone contact layer dressing that will reduce pain on removal and assist healing.


- Leaving the dressing in place will assist the wound to heal.
- This dressing can be left intact for up to 7 days depending on the wound condition and amount of ooze.
- The dressing will need to be changed if it is over 75% full.
- The dressing should be kept dry when showering.
- The dressing may be kept in place with Tubifast or a bandage.

If the wound becomes increasingly hot, painful, smelly or swollen it must be seen by your GP or Practice Nurse

*Mepilex Ag is intended for short-term use up to 4 weeks.
For long-term use, a clinical assessment by a physician is recommended.*

Special Notes:

 **Mepilex® Ag**

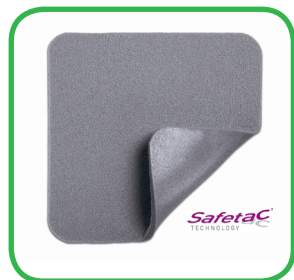

SafetaC®
TECHNOLOGY

www.molnlycke.com/patient

 **MÖLNLYCKE**
HEALTH CARE