

Minor Burn

Discharge Instructions

Care of your Mepilex® Ag Dressing

Your burn has been dressed with a silicone contact layer dressing that will reduce pain on removal and assist healing.


- Leaving the dressing in place will assist the burn to heal.
- This dressing should be left intact as per Clinician instructions.
- The dressing will need to be changed if it is over 75% full.
- The dressing should be kept dry when showering.
- The dressing may be kept in place with Tubifast™ or a bandage.

If the burn becomes increasingly hot, painful, smelly or swollen it must be seen by your hospital clinician or Outpatients Practice Nurse

*Mepilex® Ag is intended for short-term use up to 4 weeks.
For long-term use, a clinical assessment by a physician is recommended.*

Special Notes:

 Mepilex® Ag

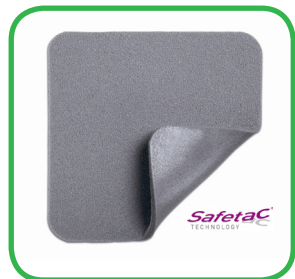

SafetaC
TECHNOLOGY

www.molnlycke.com/patient

 MÖLNLYCKE
HEALTH CARE